Reichert

**OCU-FILM® +
Tip Cover**

**Couvre-Pointe OCU-FILM® + / Copertura
per le Punte OCU-FILM® + / OCU-FILM® +
Spitzenhülle / La Cubierta de Punta
OCU-FILM® +**

Product Information and Instructions

Informations et instructions concernant le produit
/ Informazioni sul prodotto ed istruzioni per l'uso
/ Produktinformation und Gebrauchsanweisung /
Instrucciones e información sobre el producto

Instructions For Use of OCU-FILM[®] + Tip Cover with TONO-PEN[®] Tonometers

PREPARATION OF THE INSTRUMENT:

Before measuring intraocular pressure, cover the tip of the Tono-Pen with an Ocu-Film[®] + Tip Cover.

Slide Ocu-Film[®] + Tip Cover onto the Tono-Pen tip making certain that the rubber is flat across the tip, but not taut.

NOTES:

- Use of the Ocu-Film® + Tip Cover is required under original Tono-Pen instrument warranty and service contract terms.
- Corneal topical anesthetic is required with tonometry.
- Refer to the Tono-Pen instruction manual for further information.

**CAUTIONS/WARNINGS:**

- THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.
- An Ocu-Film® + Tip Cover must be applied to the Tono-Pen instrument before performing tonometry. Failure to use an Ocu-Film® + Tip Cover may result in an abrasion of the cornea.
- It is imperative that a fresh Ocu-Film® + Tip Cover be used for each patient for the protection of the eye and the instrument tip. The transfer of infection is thus prevented. Replace Ocu-Film® + Tip Covers after examination of each patient.
- Each box of high quality Ocu-Film® + Tip Covers has a “USE BEFORE” date stamped on the box. We suggest use of our Ocu-Film® + before this date to guard against the possibility of shelf wear. The latex material used in the Ocu-Film® + Tip Covers can degrade. A degraded film may result in leakage of wetting solution and instrument damage. Examine each Ocu-Film® + Tip Cover for yellowing, cracks, or a sticky texture prior to use. Ocu-Film® + Tip Covers should be stored between 35° and 80° Fahrenheit (2°-27° Celsius).

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