



## Model 2210CART Professional Medical Cart



## User Instructions

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

## TABLE OF CONTENTS

Caution and Warning.....	3
Specifications for 2210CART .....	3
Certifications / Service.....	4
Assembly Instructions 2210CART .....	5
Transportation / Operating Instructions .....	6
Maintenance.....	6
Warranty.....	7

## CAUTION AND WARNING

- To prevent patient injury, or damage to your 2210CART, please follow these instructions very carefully.
- Follow instructions and exercise caution when assembling the casters.
- Operate the 2210CART per the enclosed user instructions.
- Never transport a pediatric patient on the 2210CART.
- Do not exceed recommended weight limit of 200 lb / 90 kg for this cart.
- Do not use in the presence of flammable or explosive materials.
- If the cart becomes damaged, do not operate cart until properly serviced. All repairs shall be performed exclusively by authorized Health o meter® Professional service personnel.
- When using the 2210CART with specified scales, always attend the pediatric patient throughout the entire weighing event.
- Lock wheel casters when in storage or during a weighing event.

## SPECIFICATIONS FOR THIS 2210CART

### General

The Model 2210CART professional medical cart employs a proven and rugged mechanical design with non-conductive casters, two casters with locks, and a lockable cabinet. The 2210CART is designed with a top shelf, providing a stable, flat, and level surface area for repeatable weight measurements. In addition, the 2210CART is specifically designed for use with Health o meter Professional pediatric scales 549KL and 553KL, as well as the 2210KL Health o meter Professional neonatal scale. The 2210CART has built-in handles for an easy grip.

### 2210CART Specifications

<b>Capacity:</b>	200 lb (90 kg)	
<b>Physical Dimensions:</b>	<b>Cart (Overall Dimensions)</b> Width: 26 1/2" (67.3 cm) Depth: 18" (45.7 cm) Height: 34 1/8" (86.7 cm) Weight: 36 lb (16.50 kg)	<b>Lockable cabinet (Dimensions)</b> Width: 23" (58.4 cm) Depth: 16 3/4" (42.5 cm) Height: 17" (43.2 cm)

# CERTIFICATIONS / SERVICE

## Certification Descriptions

<b>ISO13485:2003 CERTIFIED</b>	
------------------------------------	--

## CUSTOMER SERVICE INFORMATION

The 2210CART professional medical cart is shipped assembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

# 2210CART ASSEMBLY INSTRUCTIONS

## **Before Assembly**

Each 2210CART professional medical cart is shipped assembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the cart.

To prevent scratching any components and avoid damaging the cart when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging. Carefully remove assembly from the carton and unwrap the packing materials. Set the carton aside for storage.

## **Parts List**

### **Carton**


- (1) 2210CART
- (4) Casters
- (2) Keys

## **Tools Required**

Rubber mallet (not included)

## **Assembling the 2210CART (Refer to Figure 1)**

1. Carefully remove the 2210CART cabinet assembly and casters from the carton. Place the cart on a flat, level, and dry surface. Locate and secure the two cabinet keys and four casters.
2. Position the cabinet such that the handle bar is in the upright position.
3. Insert the stems of the (2) casters with brakes in the top two holes as shown. Insert the other (2) casters in the bottom two holes. Tap casters into holes with rubber mallet until fully seated.
4. Carefully lift the 2210CART upright on all four casters. The 2210CART is now assembled and ready for use.

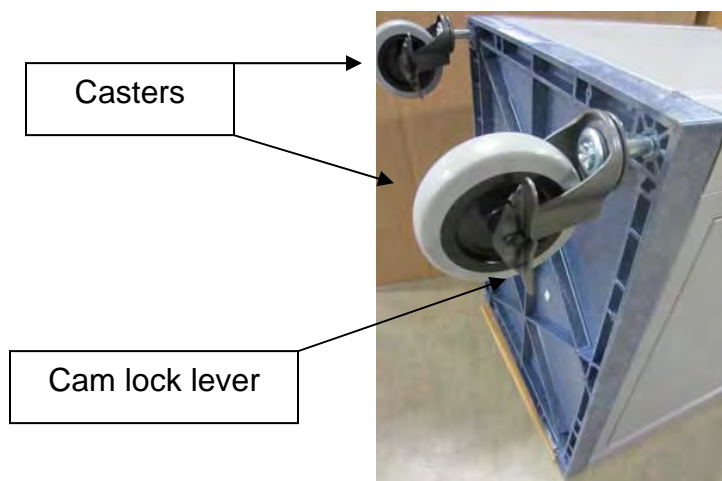

**Figure 1**

# TRANSPORTATION / OPERATING INSTRUCTIONS

## **Preparing the 2210CART for Transportation**

This cart can be easily used for transporting medical equipment or Health o meter Professional - specified scales.

**Warning: Never use the cart to transport a pediatric patient.**

To prevent unintended 2210CART movement, caster locks are provided. To engage the caster locks, step on the side of each cam lock lever marked "ON". To disengage the caster locks, step on the side of each cam lock lever marked "OFF" (see Figure 1).

## **Operating the 2210CART**

### **Lockable cabinet**

The 2210CART's lockable cabinet may be used with or without keys. The door has a positive latching action to maintain its closed position.

### **Scale Positioning Instructions on 2210CART**

For accurate weighing results, position the Health o meter Professional - specified scale on the top shelf, such that the scale is centered on the shelf.

# MAINTENANCE

Follow the instructions below for maintenance and cleaning of the 2210CART. Maintenance operations other than those described in this manual should be performed by qualified service personnel only.

## **MAINTENANCE**

Before first use and after periods of non-use, check the 2210CART for proper operation. If the cart does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the cart for any obvious damage or wear.
2. Do not store near chemicals, heaters, or radiators.
3. Do not drop, kick, or jump on the 2210CART.
4. If your cart does to work properly, do not attempt to dismantle it, as this will void your warranty. See warranty information instructions for having your 2210CART serviced.
5. Periodically inspect 2210CART casters and brakes. Remove all foreign material picked up by the casters/brakes.

## **CLEANING**

Proper cleaning is essential to ensure safe and effective operation.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the 2210CART into cleaning or other liquid solutions.
3. Do not use abrasive cleaners.