




Welch Allyn Ear Wash System

Improve efficiency and patient safety with this advanced ear wash system.

WelchAllyn[®]

Advancing Frontline Care[™]

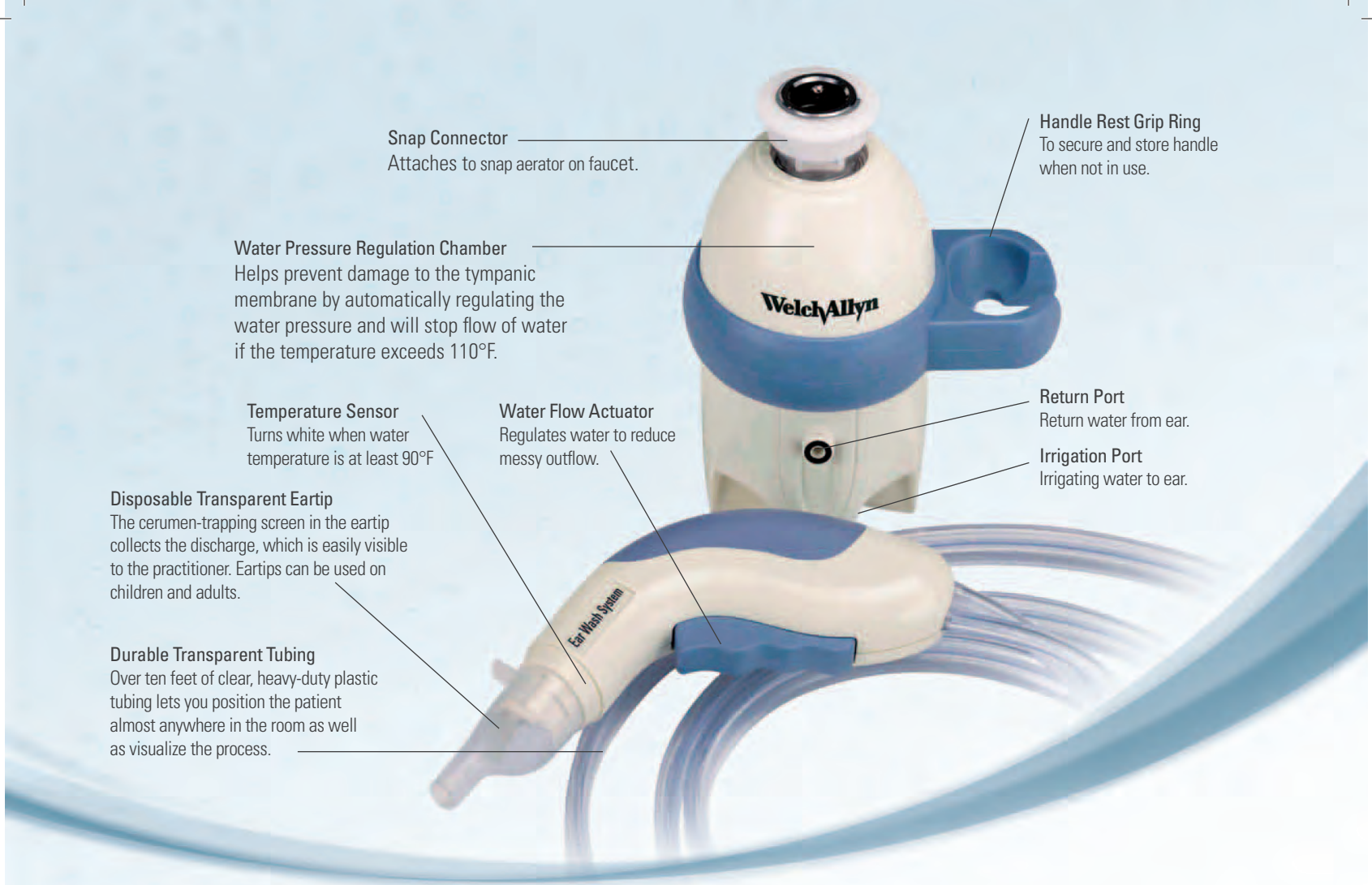

Snap Connector
Attaches to snap aerator on faucet.

Handle Rest Grip Ring
To secure and store handle when not in use.

Water Pressure Regulation Chamber
Helps prevent damage to the tympanic membrane by automatically regulating the water pressure and will stop flow of water if the temperature exceeds 110°F.

Temperature Sensor
Turns white when water temperature is at least 90°F

Water Flow Actuator
Regulates water to reduce messy outflow.

Return Port
Return water from ear.

Irrigation Port
Irrigating water to ear.

Disposable Transparent Eartip
The cerumen-trapping screen in the eartip collects the discharge, which is easily visible to the practitioner. Eartips can be used on children and adults.

Durable Transparent Tubing
Over ten feet of clear, heavy-duty plastic tubing lets you position the patient almost anywhere in the room as well as visualize the process.

- Eliminates messy outflow by automatically dislodging, removing and disposing of cerumen
- Reduces potential for common side effects such as vertigo, nausea, deeper cerumen impaction, perforated ear drums and other tympanic membrane damage
- Easily control and monitor suction, water temperature and pressure levels
- Compact and easy to transport from room to room

Welch Allyn Ear Wash System Specifications

| | |
|--------------------|--|
| Environment | Operating Temperature: 50°– +104 °F (10°– +40 °C) Storage Temperature: –40°– +120 °F (–40°– +49 °C) |
| Water | Flow Rate from Eartip: 125 mL (0.0264 gal) of water per 20 seconds at operating pressure. Source: (faucet) Water Pressure Range: 40–100 Psi (275 kPa–690 kPa) Minimum Source Water Flow Rate: 7.5 l/min (2 gal/min) Water Pressure from Eartip: Typical range 10–13 Psi (69 kPa–90 kPa) |

Ordering information

| | |
|-------|--------------------------------------|
| 29350 | Ear Wash System |
| 29360 | Eartips (case is 4 boxes of 25 tips) |
| 29330 | Tubing (hose assembly) |
| 29320 | Handle and hose assembly |
| 29372 | Snap aerator |
| 29373 | Aerator adaptor kit U.S. |
| 29381 | Aerator adaptor kit outside U.S. |



MERCOFRAMES OPTICAL CORP

5555 Nw 74 Ave. Miami, Fl 33166 305-882-0120

www.mercoframes.com sales@mercoframes.net

WelchAllyn®

Advancing Frontline Care™