



FIND, ALIGN AND CONFIRM

With just three easy movements you are able to make a full test. Grip the NETRA with both hands, palms wrapped underneath for support.

1 STEP

Turn the Brightness knob to locate the red and green umbrellas so that they are as full and bright as possible



2 STEP

Turn the Alignment knob to align the two stems of the umbrellas so that they overlap as closely as possible



3 STEP

Press the Confirm button (✓) when complete



Repeat steps 1, 2 and 3, 8X per eye until you see a blue check mark on screen.

REMOVE THE PHONE TO VIEW YOUR RESULTS.