



## Irrigating Contact Lens

**REF** TR9840-11 Flat Lens

**REF** TR9841-11 Minus Lens

### Indication For Use

Assists in visualization of surgical field while providing means for irrigation.

### Directions For Use

#### *Circulating Nurse:*

1. Peel open the outer pouch letting the inner pouch drop on to the sterile field. Use caution not to touch the inner pouch.

#### *Scrub Nurse:*

1. Peel open the sterile pouch, place irrigating contact lens onto the sterile field by turning tray up-side-down.
2. Break and remove paper tape from lens steel tube.
3. Connect female adaptor fitting of tubing to male adaptor end of irrigation solution.
4. Run irrigation solution through contact lens until all air bubbles are clear of the lens.
5. The irrigating contact lens is now ready for use.

### WARNING

This sterile disposable medical device should not be reused. After use, all parts of this disposable pack should be properly disposed of.

If any component in this pack is received in a defective condition, do not use and notify Sonomed Escalon



STERILE EO

5539 Rev. G (ECO101026)



## MERCOFRAMES OPTICAL CORP

5555 Nw 74 Ave. Miami, Fl. 33166. 305-882-0120  
ale@mercoframes.net www.mercoframes.net